

EN Operating / Care instructions Basicline 415

1. Operating instructions

Basicline 415 is a motorized height adjustable washbasin lift that is mounted under the mirror or mirror cabinet.

- Press and hold the button to start the operation.
- Release the button when the washbasin is at the desired level.

Pos. 1

Cable hand control

Pos 2.

Safety stop switch - prevents the descent of the lift in the event of an obstruction.



2. Maintenance

The Washbasin lift is maintenance-free. Greasing and other actions are made during the manufacturing for the lifetime of the lift. From safety point of view however some components shall be inspected each year.

- Inspections, service and repair work shall be made by competent persons.
- Remove any objects from the washbasin before maintenance and repair works are made.

3. Functional test

- Run the washbasin lift down and up all the way to respectively end position. Check that it moves freely, without hindrance and jarring sounds.
- Check that no cables or hoses are jammed and that the water and sewer hoses are properly fixed to the wall.
- Run the washbasin lift to the end position and make sure the hoses are long enough above the fixing point.
- Make sure the wall installation, washbasin installation and mirror installation are intact. If the frame is inclined due to wall installation, adjustments must be made.
- Check the function of safety switch:
Run down the lift while lifting the front edge of the wash basin. Make sure that the lowering is stopped.

4. Cleaning

Clean with lukewarm water and a non-scratching detergent containing soap or similar. Use a soft cloth or similar. After cleaning the surfaces shall be dried to avoid lime deposits.

**Scratching detergents or tools e.g. steel wool may not be used.
Consider that the product contains electrical components.**

5. Service & maintenance records

Order	Serial	Date
Date	Date	
Sign	Sign	
Remarks	Remarks	
Date	Date	
Sign	Sign	
Remarks	Remarks	