

# EN Operating / Care instructions Basicline 433

---

## 1. Operating instructions

Wall mounted electric washbasin lift with integrated mirror and LED lighting above.

- Press and hold the button to start the operation.
- Release the button when the washbasin is at the desired level.

### Pos. 1

Cable hand control

### Pos 2.

Safety stop switch - prevents the descent of the lift in the event of an obstruction.



## 2. Maintenance

The Washbasin lift is maintenance-free. Greasing and other actions are made during the manufacturing for the lifetime of the lift. From safety point of view however some components shall be inspected each year.

- Inspections, service and repair work shall be made by competent persons.
- Remove any objects from the washbasin before maintenance and repair works are made.

## 3. Functional test

- Run the washbasin lift down and up all the way to respectively end position. Check that it moves freely, without hindrance and jarring sounds.
- Check that no cables or hoses are jammed and that the water and sewer hoses are properly fixed to the wall.
- Run the washbasin lift to the end position and make sure the hoses are long enough above the fixing point.
- Make sure the wall installation, washbasin installation and mirror installation are intact.
- Check the function of safety switch:  
Run down the lift while lifting the front edge of the wash basin. Make sure that the lowering is stopped.

## 4. Cleaning

Clean with lukewarm water and a non-scratching detergent containing soap or similar. Use a soft cloth or similar. After cleaning the surfaces shall be dried to avoid lime deposits.

**Scratching detergents or tools e.g. steel wool may not be used.  
Consider that the product contains electrical components.**

## 5. Service & maintenance records

Order	Serial	Date
<b>Date</b>	<b>Date</b>	
<b>Sign</b>	<b>Sign</b>	
<b>Remarks</b>	<b>Remarks</b>	
<b>Date</b>	<b>Date</b>	
<b>Sign</b>	<b>Sign</b>	
<b>Remarks</b>	<b>Remarks</b>	