


EN Operating / Care instructions Basicline 401

1. Operating instructions

Washbasin bracket with manual height adjustment, at intervals of 50 mm. Mirror and lighting move in tandem with the basin.

Height adjustment:

1. Loosen the lock nuts, pos. 1, (2 pcs.).
2. Adjust to the desired height
3. Check that the hooks on the back of the back plate are placed correctly in the wall rail.
4. Lock the position with the lock nuts, pos. 1, (2 pcs.).

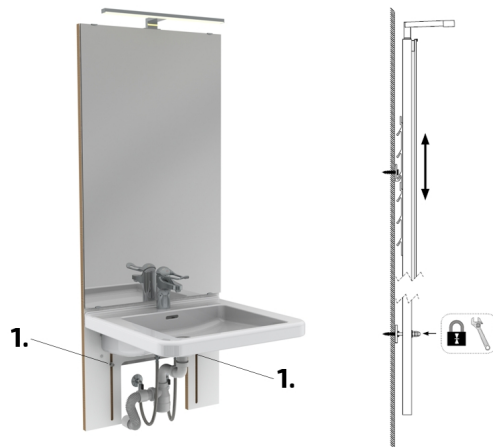

2. Maintenance

The product is maintenance-free. Greasing and other actions are made during the manufacturing for the lifetime of the lift. From safety point of view however some components shall be inspected each year.

- Inspections, service and repair work shall be made by competent persons.
- Remove any objects from the washbasin before maintenance and repair works are made.

3. Functional test

- Check that the locking nuts that lock the desired height are tightened.
- Check that no cables or hoses are jammed and that the water and sewer hoses are properly fixed to the wall.
- Make sure the wall installation, washbasin installation and mirror installation are intact.

4. Cleaning

Clean with lukewarm water and a non-scratching detergent containing soap or similar. Use a soft cloth or similar. After cleaning the surfaces shall be dried to avoid lime deposits.

Scratching detergents or tools e.g. steel wool may not be used.
Consider that the product contains electrical components (lighting).

5. Service & maintenance records

Order	Serial	Date
Date	Date	
Sign	Sign	
Remarks	Remarks	
Date	Date	
Sign	Sign	
Remarks	Remarks	