

# EN Operating / Care instructions Basicline 406


---

## 1. Operating instructions

Basicline 406 is a bracket with gas spring for height adjustment of the washbasin. The built-in gas spring helps to adjust the height of the washbasin to the desired position. No tools required for adjustment. Basicline 406 is mounted below the mirror and mirror cabinet.

### Height adjustment:

1. Pull out the locking pin and turn it 1/4. (Pos. 1)
2. Adjust the height.
3. Turn the locking pin 1/4 and release it to lock the height.

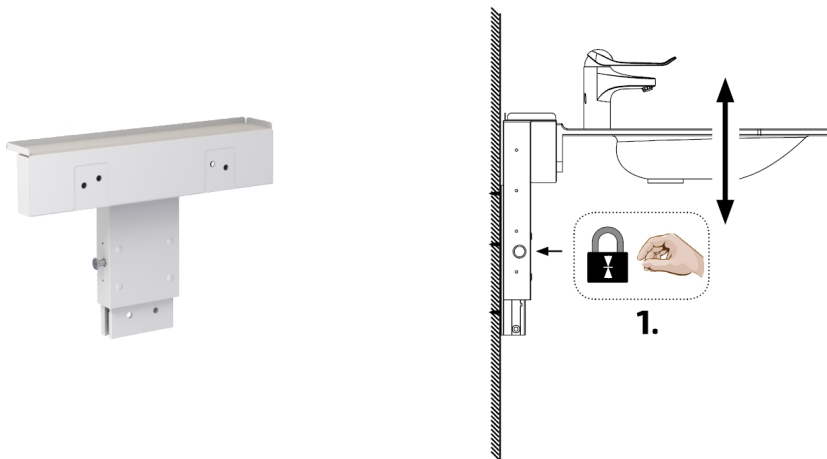

## 2. Maintenance

The product is maintenance-free. Greasing and other actions are made during the manufacturing for the lifetime of the lift. From safety point of view however some components shall be inspected each year.

- Inspections, service and repair work shall be made by competent persons.
- Remove any objects from the washbasin before maintenance and repair works are made.

## 3. Functional test

- Check that the locking pin is properly fixed to the bracket.
- Check that no cables or hoses are jammed and that the water and sewer hoses are properly fixed to the wall.
- Make sure the wall installation and washbasin installation are intact.

## 4. Cleaning

Clean with lukewarm water and a non-scratching detergent containing soap or similar. Use a soft cloth or similar. After cleaning the surfaces shall be dried to avoid lime deposits.

**Scratching detergents or tools e.g. steel wool may not be used.**

## 5. Service & maintenance records

Order	Serial	Date
<b>Date</b>	<b>Date</b>	
<b>Sign</b>	<b>Sign</b>	
<b>Remarks</b>	<b>Remarks</b>	
<b>Date</b>	<b>Date</b>	
<b>Sign</b>	<b>Sign</b>	
<b>Remarks</b>	<b>Remarks</b>	