


EN Operating / Care instructions Height adjustable table

1. Operating instructions

Height adjustable table.

- Press and hold the button to start the operation.
- Release the button (1) when the changing table is at the desired level.

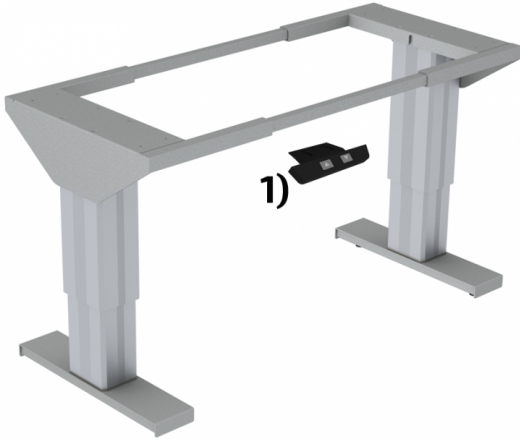

2. Maintenance

The table is maintenance-free. Greasing and other actions are made during the manufacturing for the lifetime of the lift. From safety point of view however some components shall be inspected each year.

- Inspections, service and repair work shall be made by competent persons.
- Remove any objects from the table before maintenance and repair works are made.

3. Functional test

- Minimum distance to any adjacent fitting must be at least 50 mm.
- If the distance from the changing surface to a wall or an adjacent fitting is less than 50 mm, a pinching hazard exists.
- Check that the table is firmly in place. If necessary, adjust the adjustable feet under the frame.
- Run the table down and up all the way to respectively end position. Check that it moves freely, without hindrance and jarring sounds.
- Check that no cables are jammed.
- If the table does not reach the lowest position:
 - Press the down button for 5 seconds until the control box clicks. When the control box clicks the synchronisation is ok.

4. Cleaning

Clean with lukewarm water and a non-scratching detergent containing soap or similar. Use a soft cloth or similar. After cleaning the surfaces shall be dried to avoid lime deposits.

**Scratching detergents or tools e.g. steel wool may not be used.
Consider that the product contains electrical components.**

5. Service & maintenance records

Order	Serial	Date
Date	Date	
Sign	Sign	
Remarks	Remarks	
Date	Date	
Sign	Sign	
Remarks	Remarks	