


EN Installation instructions Height adjustable table

1. Mechanical construction

1. Frame
2. Motor
3. Stand with adjustable feet

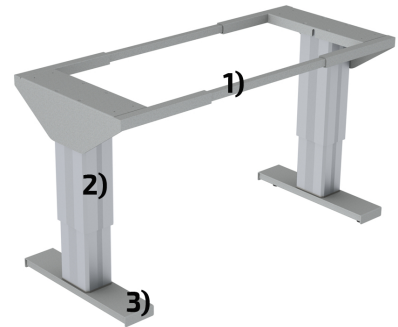

2. Delivery modules

The changing table is delivered assembled on a pallet.



3. Electrical connection

The table must be connected to a nearby main switch.

If the table is installed in a wet room, wet room regulations must be observed. The motor power supply must always be provided with an earth fault circuit breaker.

Connect the table to a 230V 50Hz power outlet. The outlet must be out of reach of children, remembering that the table surface can be raised and lowered.

Cables must be routed and fixed with care to avoid the risk of pinching.

* Power requirement 230 W.

Electrical connections must be made by a qualified electrician.

4. Location of table

Take care to locate the table so as to avoid pinching hazards.

The distance to any adjacent fitting must be at least 50 mm.

If the distance to a wall or an adjacent fitting is less than 50 mm, a pinching hazard exists.

It is the responsibility of the installer and the user to locate the table so that pinching hazards are avoided.

Adjust the feet so that the table is standing stable at the floor.

5. Functional test

- Minimum distance to any adjacent fitting must be at least 50 mm.
- If the distance from the changing surface to a wall or an adjacent fitting is less than 50 mm, a pinching hazard exists.
- Check that the table is firmly in place. If necessary, adjust the adjustable feet under the frame.
- Run the table down and up all the way to respectively end position. Check that it moves freely, without hindrance and jarring sounds.
- Check that no cables are jammed.
- If the table does not reach the lowest position:
- Press the down button for 5 seconds until the control box clicks. When the control box clicks the synchronisation is ok.