

## EN Troubleshooting Guide Height Adjustable Changing Tables

### Products

Changing Table 332  
 Changing Table 332 - Laundry sink  
 Changing Table 333  
 Changing Table 333 - Laundry sink  
 Changing Table 334  
 Changing Table 334 - Laundry sink  
 Changing Table 335  
 Changing Table 335 - laundry sink

The changing table is designed and tested for optimal reliability and service life, provided that it is operated, serviced and inspected in accordance with these instructions. Should any problem arise, consult the troubleshooting list below for guidance.

Unload the work surface prior to any troubleshooting or repairs.

Troubleshooting, service and repairs must be carried out by competent persons.

If the problem persists after taking the steps listed below, contact a qualified repair person or your supplier.

Symtom	Action
<b>The table does not move when the push-button is pressed</b>	<p>Check that the fuse for the mains feed is not damaged</p> <p>Check that the mains plug is connected.</p> <p><b>Model 334:</b> Check that the cables from the safety system and the push-button unit are properly connected to the hoist.</p> <p>Hold down the UP button for at least 20 seconds until a click is heard. Then release the button and try operating the table. If this does not help, try the same procedure with the DOWN button.</p>
<b>If the changing table does not reach the lowest position</b>	<p>Press the down button for 5 seconds until the control box clicks. When the control box clicks the synchronisation is ok.</p>